

STARR SYSTEMS LLC

DISPOSABLE AMPOULE BREAKER

U.S. Pat. Nos. 6,712,252 & 7,597,226

Our patented Disposable Ampoule Breaker (“DAB”) is a single use tool which aids the safe opening of an ampoule. The DAB prevents finger cuts and glass shards and entraps the top of the ampoule for safe disposal.

To use, simply:

Lay the ampoule on the DAB with the ampoule neck cradled into the groove at the base of the DAB.



Fold the top section of the DAB over the ampoule and snap closed.



Hold the DAB upright and check that all liquid has been removed from the tip of the ampoule.



Hold the base of the ampoule securely in one hand and gently grip the DAB in the other with your thumb on the thumb tab.



Gently pull the DAB towards thumb, snapping off the tip of the ampoule.



With the tip of the ampoule inside the DAB, snap the thumb tab shut, and dispose of properly.

SNAP, CLOSE & DISPOSE

For more information please visit our website at www.StarrSystemsLLC.com
Starr Systems, LLC, PO Box 156, McPherson, KS 67460 PH: 620-242-2377